DE	Betriebs- und Wartungsanleitung	Seite	1	009018EX-20180515
EN	Operation and maintenance manual	Page	9	
FR	Manuel d'utilisation et d'entretien	Page	16	
IT	Manuale di funzionamento e manutenzione	Pagina	23	
ES	Manual de funcionamiento y mantenimiento	Página	30	
RU	Руководство по эксплуатации и техническому обслуживанию	страница	37	

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Control device bwm 3 (electric wax knife)

Optional accessories: Rest bwm 3 - REF 14000965

Please read these instructions for use carefully before using the product!

The manufacturer cannot be held liable for any loss or damage sustained as a result of failing to comply with these instructions for use and maintenance!

Table of contents	Page
1. Product description	10
1.1 Application and use	10
1.2 Indications	10
1.3 Contraindications	10
1.4 Technical data	10
1.5 Product variants and designs	10
2. Risks and safety information	11
2.1 Warning	11
2.2 Caution	11
3. Storage and transportation guidelines	12
4. Waste disposal	12
5. Start-up	12
5.1 Device functions and components	13
5.2 Changing the modelling blade	13
5.3 Foot pedal for carrying out rapid heating	14
6. Cleaning and maintenance	14
6.1 Cleaning	14
6.2 Maintenance	14
7. Troubleshooting	14
8. Declarations of conformity	14
9. Warranty	15
10. Symbols used	15
11. Other information	15

1. Product description

1.1 Application and use

The **control device bwm 3 (electric wax knife)** is intended for use in a dental laboratory. The device is solely intended for use with conventional dental waxes. Four different sizes of attachable modelling blades make it possible to carry out a wide range of modelling applications. The temperature is controlled via a knob (potentiometer), which can be individually adapted to the different melting points of the various dental waxes.

1.2 Indications

Heating and melting dental waxes.

1.3 Contraindications

Current knowledge does not suggest any contraindications.

1.4 Technical data

Measurements:	135 / 110 / 62 mm (L/W/H)
U:	100 - 240 VAC (Input)
P:	12 W
f:	50 / 60 Hz (250 mA max.)
OUTPUT:	24 V / 500 mA
Temperature levels:	0 -10
Weight:	approx. 600 g

1.5 Product variants and designs

Control unit bwm 3 incl. mains plug	REF 14000960
Control unit bwm 3 incl. mains plug, handpiece and contouring blade size 5	REF 14000963

Optional accessories	
Foot pedal (for rapid heating)	REF 140 00961
Waxpool duo Contouring blade size 1	REF 320WP4G1
Waxpool duo Contouring blade size 5	REF 320WP4G5
Rest bwm 3	REF 14000965

2. Risks and safety information

Signal word:

Described under Points 2.1 Warning and 2.2 Caution.

Important information is highlighted using the words „Warning“ or „Caution“.

Warning

Provides information concerning potential dangers, which could result in severe injuries or death if ignored.

Caution

Provides information concerning potential dangers, which could result in injuries or damage to the device if ignored.

2.1 Warning

Damaged device

Do not use a damaged device if, for example, the housing components, cable, handpiece or the mains plug are damaged.

Improper use

- Do not heat up inflammable, explosive substances using the device!
- Do not operate or heat up in areas containing inflammable or explosive substances!
- Do not operate in areas with extreme air humidity (e.g.: steam jet, sink)
- Please do not use the device to warm up food, liquids or small live animals!
- Do not insert fingers into the device. Always unplug the mains plug first!
- Do not touch the mains plug with wet fingers, as this can lead to electric shock!
- Do not operate without work clothing or, if required, protective goggles and additional extraction (vapours due to melted wax). The relevant regulations of the German employer's liability insurance association apply.

The device must only be opened and repaired by an authorised member of the bredent service team. Turn off the device and pull the main plug prior to that!

2.2 Caution

Positioning of the device

Always ensure that the device has sufficient ventilation, otherwise it may become overheated and the service life may be impaired! The functioning of the device can only be guaranteed if the components used are in perfect condition. The components are to be thoroughly checked each time, prior to starting up the device.

Functional check/mains connection

Please ensure that...

- none of the components belonging to the device are damaged
- there are no residues or contaminants on or in the device or the accessories
- damaged accessories or components have been replaced
- the mains plug and the insulation of the power cable are not damaged in any way
- the mains plug fits into the socket and the mains data (power supply) correspond with the data on the type-plate
- the device is always positioned on a suitable flat and stable surface at work height
- the ambient temperatures move within a range of 5-40 ° C [41-104 ° F]
- the relative air humidity at 31°C [87.8°F] decreasing linearly does not exceed up to 50% relative humidity at 40°C [104°F]
- the handpiece is never operated without the attached modelling blade with handle (risk of burning)
- the handpiece is always placed on the holder on the device or the separate storage block (- risk of burning otherwise!)
- the control device bwm 3 is switched off after use.

Materials and liquids

- Water and other liquids must not be allowed to penetrate the inside of the device, as this can lead to short circuits and corrosion.
- Only use commercially-available dental materials that are processed with the control device bwm 3 (electric wax knife). Do not use any inflammable or explosive liquids or mixtures!
- Do not use any strong, corrosive or abrasive cleaning agents to clean surfaces.
- Only carry the device from A to B once it has cooled down!

Hot surfaces – risk of burning!

Hot surfaces – risk of burning! The surface of the hot needle can get hot during operation and may remain hot for a further 30 minutes after the device has been switched off. Always place a modelling blade including handle on the hot needle and place the entire handpiece in the holder [5] on the device or on the optional storage block [12].

3. Storage and transportation guidelines

The control device bwm 3 (electric wax knife) must be stored in its original packaging and in a dry area. Shipping is done in the original packaging as well, as any damages shall be borne by the sender!

4. Waste disposal

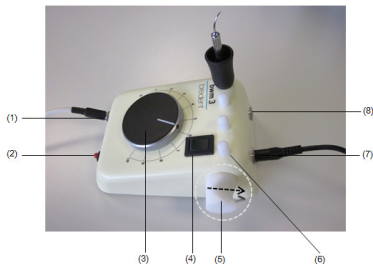
The device should be disposed of in accordance with the usual legal regulations for the disposal of waste electronic equipment in the country in question (e.g.: recycling centre, Recycling plant).

5. Start-up

Check the device for any transport damage as soon as it is removed from the packaging. Report any transport damage to brendent or an authorised dealer immediately. The control device bwm 3 (electric wax knife) may only be used by qualified personnel, such as dental technicians and dentists, or by persons being instructed and supervised by such qualified personnel. Check whether the network data complies with the data on the type-plate. Read the operation and maintenance instructions as well as any specific safety information and hazard warnings listed therein, as well as the list of device functions and components before you connect it to the mains.

5.1 List of device functions and components

- (1) - Device socket for handpiece cable
- (2) - Indicator light for foot pedal
- (3) - Knob (potentiometer), setting parameters 1 -10
- (4) - ON/OFF switch
- (5) - Holder for entire handpiece
(Insert the edge of the handle (9) beneath the hook!)
- (6) - Holding burl for attachable modelling blade (9)
- (7) - Device socket for power supply cable plug
- (8) - Device socket for foot pedal
- (9) - Modelling blade incl. handle for holder on (5)
or storage block (12)
- (10) - Hot needle (heat supply)
- (11) - Handpiece
- (12) - Storage block for handpiece

The power supply cable plug is inserted into the control device bwm 3 device socket (7). The modelling blade (9) is attached as far as the stop via the hot needle (10) and on the handpiece (11). The completed handpiece is then pushed as far as the end position on the holder on the side of the device using the handle under the retaining hook. Insert the handpiece cable into the device socket (1) intended for this purpose in the front region. Switch the main switch (4) of the device to the on = I position; the indicator light on the main switch illuminates green. Set the desired temperature on the knob (3). The temperature varies depending on the setting parameter (steps 1-10), size and shape of the blade used.

5.2 Changing the modelling blade

Modelling blade incl. handle (9)

Hot needle (10)

Handpiece (11)

The modelling blade can be changed when it is hot. In order to do this, remove the hot modelling blade from the hot needle (10) and handpiece (11) using the coloured handle (9).

Caution: Do not touch the hot needle or modelling blade – risk of burning!

Place the modelling blade (9) on one of the free holding burls (6) and move the desired modelling blade to the end position on the handpiece (11). Wax resorption can be commenced again after approx. 10 seconds.

5.3 Foot pedal for carrying out rapid heating

The rapid heating function is useful if a higher temperature is required in the short-term or if a modelling blade has been changed and needs to be brought to the right operating temperature quickly. Insert the foot pedal cable into the socket (8) on the rear of the device. If a higher temperature is required than that which has been set, the new heat output can be achieved by pressing the foot pedal. The temperature then returns to the value set. The indicator light (2) illuminates when pressing the foot pedal. Wax resorption can be commenced after approx. 10 seconds.

6. Cleaning and maintenance

6.1 Cleaning

Only clean the outside of the device using a lightly dampened cloth with the mains plug unplugged. Wipe away wax residues using a cotton cloth. Do not use any cleaning products containing solvents!

6.2 Maintenance

The device is maintenance free for the customer.

7. Troubleshooting

Fault	Cause	Troubleshooting
Device does not start/work!	network data and/or connections, parameters are not correct	- Compare data with type-plate - Insert mains plug - Set start button to ,I'
	network cable defective – cable breakage, defective contact	renew network cable
no temperature control possible	potentiometer, error in temperature control	Contact bredent service or authorised dealer and have the control panel replaced – send in device

8. Declarations of conformity

bredent GmbH & Co.KG, Weissenhorner Str. 2, 89250 Senden, Germany, hereby declares that the control device bwm 3 (electric wax knife) has been developed in accordance with the relevant provisions, European directives and harmonised standards in view of the design and construction of the product in the form brought to the market by us. The declaration of conformity can be requested if required.

9. Warranty

The control device bwm 3 (electric wax knife) has a 1-year warranty.

In order to make a claim under the warranty the original receipt of the specialist shop must be presented. Warranty services do not result in an extension of the warranty!

Faults and consequences that can be attributed to improper treatment, cleaning, maintenance and natural wear are excluded from the warranty. Any modifications or repairs to the device and its components must only be carried out by bredent or by third parties with express authorisation (certification) to carry out such work, and those modifications or repairs must be carried out in line with legal engineering practice. Failure to comply with the above will result in the invalidation of the warranty! The product must only be used by dentists, dental technicians and specially trained experts. Only original tools and parts are to be used for processing. The serial numbers for all parts used must be documented to ensure traceability and help deal with any complaints. These user instructions reflect the current state of the art and our own experiences. The product may only be used for the indication described under Point 1.2.

10. Symbols used

Typ	control device bwm 3 (electric wax knife)
REF	Batch code
SN	Serial number
BJ	Year of manufacture
	Manufacturer
	European conformity marking
	Follow the instructions!
	Caution
	Hot surfaces!
	Unplug the device from the mains supply prior to opening it!
	Do not dispose of with household waste

11. Other information

The information contained in these instructions for use is always updated in line with the latest results and experiences. We therefore recommend reading the instructions for use prior to using a new device. The user is responsible for how the product is used. No liability is accepted for defective results, since bredent has no influence on subsequent workmanship. Any claims for compensation may only be based on the value of our products.

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