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Schraubendreher

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Instructions for use

Screwdriver

Before using the product, please read these instruction for carefully use. The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

The screwdrivers are available in various types and used for screws from the product range of bredent. **Important:** The screwdrivers need to be sterilized prior to the first oral use.

2. Indication

The screwdrivers are suitable for use in the laboratory and for oral use. They are used to tighten or loosen screws when inserting or removing partially removable dentures. Screws to adjust the friction of removable dentures can also be activated or deactivated by the dentist.

Important: The maximum torque of 20 Ncm must be observed when tightening the bredent screws!

The screwdrivers are used for the following products:

Screwdriver, long, screwdriver, short, screwdriver, contra-angle, screwdriver set:

- Security-Lock system
- Friction Splint FS1
- Set of tools for individual screw connections 1.4 and 1.6
- Bridge sectioning attachment oc
- Screw set, partly preassembled
- Vario-Soft 3 bridge sectioning attachment
- SKY fast & fixed prosthetic coping
- Stud screws vks-oc / sg exchangeable stud 2.2
- Vario Compress 1 and Vario Compress 2

- Vario-Stud-Snap vks-oc rs 2.2 mm abutment

Screwdriver stud screw

- Stud screws vks-oc / sg exchangeable stud 17

3. Contraindication

Based on the current state of knowledge there are no known contraindications. The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Hazard and safety information

The instructions on repeated treatment (cleaning/disinfection) of the manufacturers of the sterilization devices must be adhered to in order to avoid cross contamination.

In cases of oral use, the screwdrivers must be protected against or swallowing with a chain or dental floss fastened on the wrist of the user.

5. Storage and durability information

The product is supplied in the non-sterile condition and dry and dust-free storage must be ensured. The product must only be stored in a clean and sterilized condition in accordance with the instructions of the manufacturer of the sterilization device.

6. Use

6.1 Initial and repeated treatment

For treatment purposes (cleaning/disinfection), all parts assembled must be disassembled into their individual components immediately after the use and prior to first use and cleaned manually under cold, running water using a brush. Then disinfection is carried out in a disinfectant bath (e.g. Komet DC1). The instructions of the manufacturer must be adhered to. Then the instruments are rinsed three times with distilled water and dried carefully using lint-free disposable cloths. Use oil-free compressed air to dry drill holes. Checks for damage, corrosion and proper function are required. The individual components are assembled and all instruments and tools are placed into the respective positions in the tray. The instruments and tools can be sterilized using steam sterilization in the vacuum procedure. For this purpose, three-times fractionated pre-vacuum must be generated with a sterilization time of 4 minutes and a temperature of 134 °C ± 1 °C and a pressure of 2 bar.

Designation	Manufacturer
D20 Drill Aktiv	Schülke & Mayr
Stammopur DB	Dr. Stamm GmbH
Bechtol-Bohrerbad	Becht
Korsolex Bohrerbad	Bode
DC1	Komet
Dürr Systemhygiene ID 212/220	Dürr Dental
Septanin Bohrerbad	Gaba
Sekudrill	Ecolab
Micro 10plus	Unident
M + W Bohrerbad neu	Müller + Weygandt GmbH
Kentocid	Kent Dental
Grotanat-Bohrerbad	Schülke & Mayr

6.2 Torques

The specified torques of max. 20 Ncm must not be exceeded and the indication range or application areas of the tools must be observed (see instructions). Non-compliance with the instructions may result in damage or even in fracture of the instruments or screws, which may cause aspiration or swallowing of the fragments or injury to the patient.

7. Technical data

REF	Product	Material
330 0081 2	Screwdriver, long	Surgical steel
330 0069 0	Screwdriver, short	Surgical steel
330 0081 3	Screwdriver, contra-angle	Surgical steel
330 0081 0	Screwdriver set	Surgical steel
330 0116 4	Screwdriver. stud screw	Surgical steel
460 0001 0	Screwdriver is, contra-angle	Surgical steel
460 0001 1	Screwdriver is, manual, short	Surgical steel

8. Symbols

	Lot number
	Observe accompanying documents
	Non sterile
	Dry storage

9. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff. Only original tools and components may be used for processing.

Prior to the integration into the patient's mouth, all components must be disinfected. Only standard disinfectants for dental instruments may be used, e.g. Dürr ID 212 forte. The manufacturer's instructions on the use of the disinfectant must be observed.

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on processing. Any occurring claims for damages may only be made up to the value of our products.

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GmbH & Co.KG · Weissenhorner Str. 2 · 89250 Senden · Germany