DE	Betriebs- und Wartungsanleitung	Seite	1	009269EX-20180417
EN	Operation and maintenance manual	page	9	
FR	Manuel d'utilisation et d'entretien	page	16	
IT	Manuale di funzionamento e manutenzione	pagina	23	
ES	Manual de funcionamiento y mantenimiento	pagin	30	
RU	Руководство по эксплуатации и техническому обслуживанию	страница	37	

Waxpool duo

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Waxpool duo

Please read these instructions for use carefully before using the product!

The manufacturer cannot be held liable for any loss or damage sustained as a result of failing to comply with these instructions for use and maintenance!

Table of contents	Page
1. Product description	10
1.1 Application and use	10
1.2 Indications	10
1.3 Contraindications	10
1.4 Technical data	10
1.5 Product variants and designs	11
2. Risks and safety information	11
2.1 Warning	11
2.2 Caution	11
3. Storage and transportation guidelines	12
4. Waste disposal	12
5. Start-up	12
5.1 List of device functions and components	13
5.2 Digital display	13
5.3 Changing from °C to °F	13
5.4 Waxpool duo dip tanks	13
5.5 Electric wax knife	14
6. Cleaning and maintenance	14
6.1 Cleaning	14
6.2 Maintenance	14
7. Troubleshooting	14
8. Declarations of conformity	14
9. Warranty	15
10. Symbols used	15
11. Other information	15

1. Product description

1.1 Application and use

The **Waxpool duo** is intended for use in a dental laboratory. The device is solely intended for use with conventional dental waxes. Two dip tanks can be heated up separately and modelled at the same time using the wax knife that can be attached from the side.

What is more, there is the option of also using the dip tanks as deep-drawing tanks for the production of wax copings, due to the fact that the tank has a depth of 27 cm [10.63 inches].

The dip tanks have a temperature range of 20–120°C [68–248°F] and are suitable for processing light waxes.

1.2 Indications

Heating, melting and shaping dental waxes.

1.3 Contraindications

Based on the current state of knowledge there are no known contraindications.

1.4 Technical data

Measurements:	150 / 109 / 55 mm (L/W/H)
U:	100 - 240 VAC (Input)
P:	34 W
f:	50/60 Hz (800 mA max.)
OUTPUT:	24 V / 1.5 A
Dip tank temperature	20-120°C / 68 - 248°F
Wax knife temperature	0-240°C
Display steps:	0 / 10 / 100

1.5 Product variants and designs

Waxpool duo	REF 11001500
Waxpool duo Handpiece	REF 11001510

2. Risks and safety information

Signal word:

Described under Points 2.1 Warning and 2.2 Caution.

Important information is highlighted using the words „Warning“ or „Caution“.

Warning

Provides information concerning potential dangers, which could result in serious injuries or death if ignored.

Caution

Provides information concerning potential dangers, which could result in injuries or damage to the device if ignored.

2.1 Warning

Damaged device

Do not use a damaged device if, for example, the display, housing components or the mains plug are damaged.

Improper use

- Do not to heat up any flammable, explosive substances with the unit !
- Do not operate or heat in the vicinity of flammable and explosive substances!
- Do not operate in areas with extreme air humidity (e.g.: steam jet, sink)
- Please do not use the device to warm up food, liquids or small live animals!
- Do not insert fingers into the device. Always remove the mains plug beforehand!
Risk of electric shock!
- Do not operate without work clothing and, if necessary, protective goggles. The relevant regulations of the German employer's liability insurance association apply.

The device must only be opened and repaired by authorised bredent service personnel – remove mains plug beforehand!

2.2 Caution

Positioning of the device

Operate the device at a distance of 10 cm from the wall. If the device does not have sufficient ventilation, it may become overheated and the service life may be impaired!

The functioning of the device can only be guaranteed if the components used are in perfect condition. The components are to be thoroughly checked each time, prior to starting up the device.

Functional check/mains connection

Please ensure that...

- none of the components belonging to the device are damaged,
- there are no residues or contaminants on or in the device or the accessories,
- damaged accessories or components have been replaced,
- the mains plug and the insulation of the power cable are not damaged in any way,
- the mains plug fits into the socket and the network data corresponds with the data on the type-plate
- the ambient temperatures are in a range of 5–40°C [41–104°F]
- the relative air humidity at 31°C [87.8°F] decreasing linearly does not exceed up to 50% relative humidity at 40°C [104°F]
- the device is always positioned on a suitable flat and stable surface at work height,
- the dip tanks are not overfilled with wax or inclined – this can result in leakage
- Waxpool duo is not cleaned with compressed air,
- the device is switched off after use.

Materials and liquids

- Water and other liquids must not be allowed to penetrate the inside of the device, as this can lead to short circuits and corrosion.
- Please only use commercially-available dental materials that are processed in Waxpool duo. Do not use any inflammable or explosive liquids!
- Do not use any strong, corrosive or abrasive cleaning agents to clean surfaces.
- Only carry the device from A to B once it has cooled down and the wax is solid!

Hot surfaces – risk of burning!

The surface of the Waxpool duo can get hot during operation and may remain hot for a further 30 minutes after the device has been switched off. Always place a modelling blade on the wax knife. The heating element can also get very hot.

3. Storage and transportation guidelines

The Waxpool duo must be stored in a dry area and in its original packaging. The unit is supplied in its original box, since any damage that occurs would be attributable to the buyer if this was not the case!

4. Waste disposal

The device should be disposed of in accordance with the usual legal regulations for the disposal of waste electronic equipment in the country in question (e.g.: recycling centre, Recycling plant).

5. Start-up

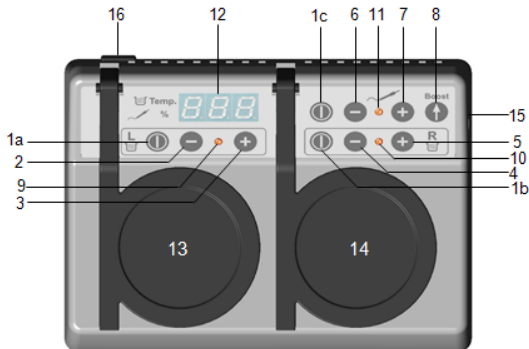
Check the device for any transport damage as soon as it is removed from the packaging. Report any transport damage to bredent or an authorised dealer immediately.

The Waxpool duo may only be used by qualified personnel, such as dental technicians and dentists, or by persons being instructed and supervised by such qualified personnel.

Insert the cable of the mains plug into the rear (16) of the device and connect to a power supply compatible with the details on the type-plate. The device carries out a self-test, where the indicator lights (9, 10 and 11) and the digital display (12) briefly illuminate. The current setting of °C or °F is displayed.

5.1 List of device functions and components

- | | |
|--------------|---------------------------------|
| 1a, 1b, 1c | Start button |
| 2, 4 and 6 | Temperature reduction |
| 3, 5 and 7 | Temperature increase |
| 8 | Boost button |
| 9, 10 and 11 | Indicator light |
| 12 | LED display |
| 13 and 14 | Dip tanks |
| 15 | Connection socket for wax knife |
| 16 | Connection for mains cable |

5.2 Digital display

The digital display always shows the value of the working part that is activated at that moment. If the indicator light is flashing, the corresponding working part is heating up; if the light is steady, the set temperature has been reached.

5.3 Changing from °C to °F

The unit currently set (°C or °F) is displayed during the self-test and remains visible for approximately 5 seconds. The unit can be changed by pressing the button combination 2 and 5 for 5 seconds.

5.4 Waxpool duo dip tanks

Fill the dip tanks (13 and 14) of the Waxpool duo until they are 2/3 full of wax.

The keypads with the border are supplied for heated tanks. Press the „I“ (1a and 1b) button for one second in order to commence the dip tank heating process; the last setting is displayed, the indicator light flashes. The temperature indicator on the display goes out after 3 seconds. Set the desired temperature using the „-“ (2 and 4) and „+“ (3 and 5) buttons. Once the temperature has been reached, the indicator light remains on constantly.

The dip tanks can be heated up to 120°C (248°F); simply press and hold the „+“ button. In order to lower the temperature, simply press and hold the „-“ button.

5.5 Electric wax knife

Insert the cable of the electric wax knife into the socket (15) intended for this purpose at the side. Start the wax knife heating process by pressing and holding the „I“ (1c) button for one second; the value that was last set will be displayed. A temperature indicator is not present here, just a value of 0 to 100. The value 100 corresponds with a temperature of approx. 240°C, depending on the blade thickness. Set the desired temperature using the „-“ and „+“ (6 and 7) buttons. If the highest possible temperature has not been set, the final temperature can be set by pressing the „▲“ button (boost button, 8). This maintains the temperature for 20 seconds and then automatically sets it back to the last value.

6. Cleaning and maintenance

6.1 Maintenance

Only clean the outside of the device using a lightly dampened cloth with the mains plug unplugged. Wipe any overflowing wax and the dip tanks with a cotton cloth. The lids of the dip tanks can be removed and cleaned when in an open position. Do not use any cleaning products containing solvents!

6.2 Maintenance

The device is maintenance-free for the customer.

7. Troubleshooting

Error	Cause	Elimination of error
Device does not work/start! LED display is not lit!	Mains data and/or connections, incorrect parameters	- Compare data with type plate - Plug in power plug - Set start key to „I“
	Defective power cord - cable broken, loose contact	Replace power cord
Keys of control panel without function	Defective contact	Contact bredent service or authorized dealer and have the keypad replaced - Send in the device.
„E1“	Overcurrent	The handpiece needs to be exchanged.
„E2“ and „E3“	Short circuit or open line	Contact service of bredent or authorized dealer and return the device for testing.
„E4“	Soll-Temperatur wurde nicht rechtzeitig erreicht	Mit dem Service von bredent oder autorisierten Händler in Verbindung setzen und Gerät zur Prüfung einschicken.

8. Declarations of conformity

bredent GmbH & Co.KG, Weissenhorner Str. 2, 89250 Senden, Germany, hereby declares that the Waxpool duo has been developed in accordance with the relevant provisions, European directives and harmonised standards in view of the design and construction of the product in the form brought to the market by us. The declaration of conformity can be requested if required.

9. Warranty

The Waxpool duo device has a 1-year warranty.

In case of any claims for warranty the original dealer's invoice is required.

Warranty services do not result in an extension of the warranty!

Faults and consequences that can be attributed to improper treatment, cleaning, maintenance and natural wear are excluded from the warranty.

Any modifications or repairs to the device and its components must only be carried out by bredent or by third parties with express authorisation (certification) to carry out such work, and those modifications or repairs must be carried out in line with legal engineering practice. Failure to comply with the above will result in the invalidation of the warranty!

The product must only be used by dentists, dental technicians and specially trained experts. Only original tools and parts are to be used for processing.

The serial numbers for all parts used must be documented to ensure traceability and help deal with any complaints.

These user instructions reflect the current state of the art and our own experiences. The product may only be used for the indication described under Point 1.2.

10. Symbols used

	Waxpool duo
	Batch code
	Serial number
	Year of manufacture
	Manufacturer
	European conformity marking
	Follow the instructions!
	Caution
	Hot surfaces!
	Unplug the device from the mains supply prior to opening it!
	Do not dispose with household waste!

11. Other information

The information contained in these instructions for use is always updated in line with the latest results and experiences. We therefore recommend reading the instructions for use prior to using a new device.

The user is responsible for how the product is used. No liability is accepted for defective results, since bredent has no influence on subsequent workmanship. Any claims for compensation may only be based on the product value.

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