Master-Pin Radix-S

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Ausgabe: 07.04.08/0D

Master-Pin Radix-S

Before using the product, please read these instructions for careful use.

1. Indication range

Master-Pin Radix-S is a plug-type pin made of high tech plastic for direct placement and fixation in the impression material. This pin is used for precise repositioning of sawed individual dies on the model base of the sawcut model.

2. Processing

2.1 Disinfecting and cutting the impression

The impression is disinfected in the usual way and then optimized by conical cutting.

2.2 Positioning and placing the Master-Pin Radix-S pins

Due to the root-like corporeal shape of the pin shaft, this pin can be very precisely positioned in the impression and fixed with the tip by pressing the pin into the impression material. Press the pin approx. 2 to 4 mm into the impression. The special shape of the Radix-S retention area - analogous to a lower anterior die prepared in a reduced size - allows to use the pin also in impressions with limited space. One Master-Pin Radix-S is used for each prepared tooth. The pin must be placed so that the thin part of the pin always faces the lingual resp. palatal direction and the thicker part the buccal resp. labial direction. This way systematic working is ensured and the dies can be sawed more easily later on.

Tip: Place two more pins in the remaining area of the impression so that the impression can safely rest (upside down) on at least three or more pins without tilting. Accordingly, subsequent preparation of the base of the arch is simplified and useful functional models are obtained.

2.3 Pouring out the arch

The arch and the inserted pins are poured out with super-hard stone. The correct height is ensured when a layer of stone with a thickness of at least 1.5 mm covers the root-shaped pin shaft - extending beyond the retention area. This way perfect thickness of the arch is achieved - as thin as possible and yet sufficiently stable. While pouring out, fill in the liquid stone so that a horizontal level is obtained. Insert retention rings (REF 310 0011 1) up to half of their height into areas which do not have to be removed; this must be done before the stone has hardened.

2.4 Separating

Apply Master-Sep plaster-plaster separating liquid (REF 520 0029 0) onto the arch and the protruding Master-Pin Radix-S pins using a brush.

2.5 Preparing the base of the arch with integrated pins

The base of the impression with the prepared arch is prepared in the usual way. We recommend to use the Master-Split system (REF 360 0124 M) for this process.

2.6 Trimming

Once the stone has hardened, the model must be trimmed circularly to a distance of approx. 1 to 2 mm to the buccal resp. labial surfaces of the teeth.

2.7 Sawing and preparing the dies

Cut off the Radix-S plastic tips which project from the occlusal area of the plaster dies using a hot instrument. The dry model is sawed as usual and the dies are ground. Finally, remove plaster dust and particles from the model base and the dies.

3. Other information

The information contained in these instructions for use is always updated according to the latest knowledge and experience. Therefore, we recommend reading the instructions for use again before using a new package. Any recommendations concerning the use are based on bredent's experience and tests and can only be considered to provide approximate values. Our products are subject to continuous further development. Therefore, we reserve the right to perform changes in the design and composition.

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