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
Bending pliers for Adams clasps

GB

Before using the product, please read these instructions for careful use.

1. Indication range

The bending pliers for Adams clasps serve to prepare orthodontic retaining elements in the form of Adams clasps made of resilient clasp wire. The bending pliers allow to produce different sizes of Adams clasps.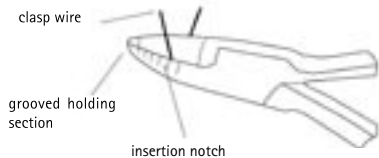

2. Use





2.1 Determining the clasp size

For premolar clasps and small molar clasps use the front insertion notches that are facing the grooved holding section. For molars use the insertion notches that are facing the handle of the pliers.

2.2 Bending the clasp

① Hold the pliers with the grooved side facing downward and open them. Insert the clasp wire into the selected notch. Close the pliers and press together. Bend the clasp wires that stand out from the sides upward (U-shaped)


② Bend the high ends of the clasp downward on the plier flanks. An M-shaped clasp is obtained.

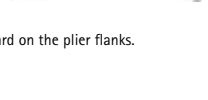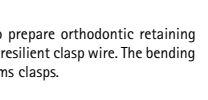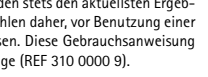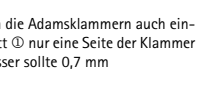③ Bend arch-shaped loops backward with the help of the grooved holding section.

④ Bend the supports of the clasp rests backward by 90° on both sides.

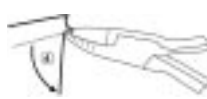⑤ The appendix of the clasp which will be situated in the resin later on is adapted to the jaw area using the grooved holding section.

3. Additional application fields

In case of limited gingival space, the Adams clasps can also be bent on one side. After carrying out step j, only one side of the clasp is shaped further. The diameter of the wire clasp should not exceed 0.7 mm.

4. Other information

The information contained in these instructions for use is always updated according to the latest knowledge and experience. Therefore we recommend to read the instructions for use again before using a new package. These instructions for use refer to the bending pliers for Adams clasps, (REF 310 0000 9).

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