DE	Betriebs- und Wartungsanleitung	Seite	1	009035EX-20180425
EN	Operation and maintenance manual	page	11	
FR	Manuel d'utilisation et d'entretien	page	20	
IT	Manuale di funzionamento e manutenzione	pagina	30	
ES	Manual de funcionamiento y mantenimiento	pagina	40	
RU	Руководство по эксплуатации и техническому обслуживанию	страница	50	

ecovac (Vakuumanrührsystem)

REF 14000930

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ecovac (Vacuum mixing system)

REF 14000930

Please read these instructions for use carefully before using the product!

The manufacturer cannot be held liable for any loss or damage sustained as a result of failing to comply with these instructions for use and maintenance!

Table of contents	Page
1. Product description	12
1.1 Application and use	12
1.2 Indications	12
1.3 Contraindications	12
1.4 Technical data	12
1.5 Product variants and designs	13
2. Risks and safety information	14
2.1 Warning	14
2.2 Caution	14
3. Storage and transportation guidelines	15
4. Waste disposal	15
5. Start-up	15
5.1 Assembly	15
5.2 Lists of device functions and components	16
5.3 Preparatory work	17
5.4 Mixing cups	17
5.5 Filling level of the mixing cups	18
6. Cleaning and maintenance	18
6.1 Cleaning	18
6.2 Maintenance	18
7. Troubleshooting	18
8. Declaration of conformity	19
9. Warranty	19
10. Symbols used	19
11. Other information	19

1. Product description

1.1 Application and use

Thanks to the matched system components and the sophisticated technology, the ecovac mixing system ensures consistent results when processing investment materials, plasters and silicones. Consistent results are achieved by the special geometry of the mixing spiral in combination with the mixing cup and the unit functions that can be adjusted individually. In the standard version, the device is suitable for wall mounting. The switches and panels are clearly arranged to enable user-friendliness. Two setting parameters for a specific vacuum allow perfect use of the materials to be mixed. The high-performance and maintenance-free pump generates the selected vacuum very quickly. The mixing cup is drawn in and held automatically at the device. The device is efficient due to the fact that it can be operated using six different mixing speeds.

1.2 Indications

Homogeneous, bubble-free mixing of dental plasters, investment materials and silicones under vacuum. Please observe the information regarding rpm, mixing times and vacuum settings of the material manufacturer and use a separate mixing cup for each material - to avoid obtaining incorrect mixing results!

1.3 Contraindications

Based on the current state of knowledge there are no known contraindications.

1.4 Technical data

Dimensions in mm	[D/W/H]:	250 x 200 x 290 (wall mounting) 250 x 250 x 650 or 590 (incl. stand)
Weight	[kg]:	13 (wall mounting) 25 (incl. stand)
Speed	[rpm]:	250 / 320 / 390 / 460 / 530 / 600
U	[VAC]:	200 - 240 (Input)

P	[W]:	220 W
f	[Hz]:	50 - 60
Fuse	[A]:	T 2.0 A

1.5. Product variants and designs

Ecovac vacuum mixing system 230 V 7 Parts (scope of delivery)	REF 14000930
1x ecovac vacuum mixing device for wall mounting	
1x mains cable, 2 m in length	
1x replacement filter	
1x sound absorber	
1x drilling template	
3x screws with dowels for wall mounting	
1x screw with floor bracket	
1x instructions for use	
Optional accessories	
Base for ecovac vacuum mixing system	REF 21000450
Ecovac mixing cups 50 ccm	REF 1400B945
Ecovac mixing cups 250 ccm	REF 1400B940
Ecovac mixing cups 750 ccm	REF 1400B942
Ecovac mixing cups 1000 ccm	REF 1400B943
Ecovac mixing spiral 50 ccm	REF 1400R945
Ecovac mixing spiral 250 ccm	REF 1400R940
Ecovac mixing spiral 750 ccm	REF 1400R942
Ecovac mixing spiral 1000 ccm	REF 1400R943
Spare parts	
Prefilter (2 pieces)	REF 73000180
Ecovac - O-ring for mixing spiral, 50 ccm	REF 99300007
Ecovac - O-ring for mixing spiral, 250 ccm	REF 99300008
Ecovac - O-ring for mixing spiral, 750 - 1000 ccm	REF 99300013
Ecovac - O-ring for mixing/suction connection	REF 99300014
Ecovac unit silencer for 14000930	REF 99300015

2. Risks and safety information

Signal word:

Described under Points 2.1 Warning and 2.2 Caution.

Important information is highlighted using the words „Warning“ or „Caution“.

Warning

Provides information concerning potential dangers, which could result in severe injuries or death if ignored.

Caution

Provides information concerning potential dangers, which could result in injuries or damage to the device if ignored.

2.1 Warning

Damaged device

Do not use a damaged device if the housing components, mixing cup, mixing spiral, stand, cable or the mains plug are damaged.

Improper use

- Do not heat flammable, explosive substances with the device!
- Do not operate or heat in the vicinity of flammable and explosive substances!
- Do not operate directly in areas with extreme air humidity (e.g.: steam jet, sink)
- Do not use on patients or animals (not a medical device!)
- Do not insert fingers into the device. Always unplug the mains plug first!
- Do not touch the mains plug with wet fingers, as this can lead to electric shock!
- Do not operate without work clothing and suitable protective goggles. The relevant regulations of the German employer's liability insurance association apply.

The device must only be opened and repaired by authorised bredent service personnel – remove mains plug beforehand!

2.2 Caution

Positioning of the device

- Always ensure that the device has sufficient ventilation, otherwise it may become overheated and the service life may be impaired!
- Only use in closed, dry rooms and on a stable, level surface (workstation) with a minimum storage area of 50 x 50 cm!
- The functioning of the device can only be guaranteed if the components used are in perfect condition. The device and its attachments are to be thoroughly checked each time, before it is started up!
- Tools must only be changed once the device has been switched off!
- Never operate without the mixing spiral and the mixing cup!
- Never operate the vacuum pump without the suctioned mixing cup!
- Do not touch the running mixing/suction connections (6)!
- Hold the mixing cup firmly after ending/reducing the vacuum!

Functional check/mains connection

Please ensure that:

- none of the components belonging to the device are damaged
- there are no residues or contaminants on or in the device or the accessories
- damaged accessories or components have been replaced
- the mains plug and the insulation of the power cable are not damaged in any way
- the plug of the device cable is compatible with the socket and the mains data (power supply) are in compliance with the information specified on the type plate
- the device is always positioned on a suitable flat and stable surface at work height and set up on the stand or securely mounted on the wall

- the ambient temperatures are in a range of 10–40°C [50–104°F]
- a relative humidity of 80% is not exceeded
- you observe the manufacturer's information relating to rotating tools
- the ecovac is switched off after use

Device fuse – changing the microfuse

In the event that it is no longer possible to switch on the device, please check that the fuse is working correctly. Switch the device off first and remove the power cable from the mains!

Do not loosen the screws of the mains plug!

The compartment for the fuses is located in the middle between the mains plug (5) and the cable connection of the and can be removed if the device is switched off and unplugged (use an instrument/tool for easier removal). Check whether the metal filament of the removed fuse is broken. Then reinsert the fuse holder (with the new fuse) into the unit. Switch the device on and check proper function – if keys or display are not lit and/or the vacuum pump or the motor are not running, please send the unit to bredent or an authorized dealer.

Materials and liquids

- Water and other liquids must not be allowed to penetrate the inside of the device, as this can lead to short circuits and corrosion.
- Only use commercially-available dental materials that are processed with the ecovac.
- Do not use any inflammable or explosive liquids or mixtures!
- Do not use any strong, corrosive or abrasive cleaning agents to clean surfaces.
- Only carry the device from A to B when wearing safety shoes!

3. Storage and transportation guidelines

The ecovac mixing device must be stored in a dry area and in its original packaging. The device is also shipped in the original packaging since damage to the device caused by a different packaging (box) will be at the expense of the shipper.

4. Waste disposal

The device should be disposed of in accordance with the usual legal regulations for the disposal of waste electronic equipment in the country in question (e.g.: recycling centre, Recycling plant).

5. Start-up

Check the device for any transport damage as soon as it is removed from the packaging. Report any transport damage to bredent or an authorised dealer immediately. The ecovac mixing device may only be used by qualified personnel, such as dental technicians and dentists, or by persons being instructed and supervised by such qualified personnel.

5.1 Assembly

Wall mounting

The drilling template also supplied makes it easier to carry out accurate assembly. The drilling template must be horizontally aligned using a spirit level. The mixing/suction connections (6) should be positioned at least 30 cm from the tabletop (mixing cup slot). After drilling the three holes (diameter of 8 mm), press the dowels in and screw in the top two screws so that they protrude by approx. 6 mm. The bottom bracket is fastened with the screw in the lower „hole“ (10) at the device. Then the device can be attached to the wall and the bottom bracket is screwed to the wall. Safe hold of the device must be ensured. Also correct the top two screws, if necessary. Please ensure that the wall is stable and allows for the device to be mounted.


Assembly as a floor-mounted appliance (optional)

The stand REF 21000450 is attached to the rear of the device using the screws provided. There are two different assembly heights available. Place the device on a level surface and ensure it has a secure position.


5.2 Lists of device functions and components

- | | |
|--------------------------------------|--|
| (1) Speed regulator | (increments: 290 – 600 rpm) |
| (2) Time display | (min./sec; 0:00 – 9:50) |
| (3) Time setting | (-/to reduce+/to increase) |
| (4) Mixing function START/STOP | (Programme sequence) |
| (5) ON/OFF | (mains plug, bottom right of the device) |
| (6) Mixing/suction connections | (stirrer-cup slot) |
| (7) Vacuum I = START / 0 = STOP | (vacuum pump) |
| (8) Vacuum display | (I=partial vacuum / II=full vacuum) |
| (9) Vacuum changeover switch | (change of position I or II) |
| (10) Lower „hole“ for bottom bracket | |
| (11) Schalldämpfer | |
| (12) Typenschild | |

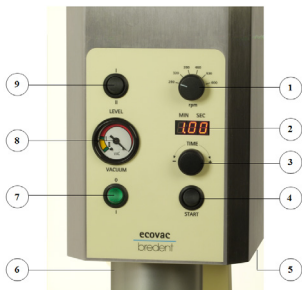Check whether the mains data are identical with the information specified on the type plate (12). Before connecting the device to the mains supply, read the operation and maintenance instructions carefully, in particular the safety and hazard information, information on assembly and list of device functions and components provided in the instructions! The silencer (11) is not assembled to the device for reasons of transport safety. It is assembled to the unit before initial operation.

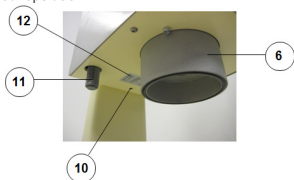5.3 Preparatory work

Select the mixing cup with the appropriate filling quantity (see 5.5) for the material to be mixed. The filling quantity must not be exceeded, as otherwise there is the risk that the material will be sucked in too high by the vacuum. This could result in contamination of the prefiler. Mix the material that requires mixing in accordance with the manufacturer's information in the instructions for use. Ensure that the edge of the cup is free of contaminants, as the lid of the mixing spiral will otherwise not close fully and a vacuum will therefore not be generated. Switch on the main switch (5) on the bottom right of the device. The time for the mixing process will be stated by turning the adjustment knob (3) and will be displayed on the digital time display (2). The desired speed is set on the speed regulator (1).

Please note:

The expansion of the investment material varies depending on the speed set. 390 rpm is recommended as a base value. Vacuum I or II is selected on the changeover switch (9). Fit the closed cup in the mixing/suction connections (6) by turning it. The vacuum is commenced by moving the toggle switch (7) to the „I“ position; you should continue to hold the cup for around 5 more seconds. Once the vacuum has built up, the cup will remain in place by itself. The vacuum setting can be changed from I to II (9) whilst the vacuum is building up. If the vacuum has been set to I, the pump will switch off once the vacuum has been reached; the vacuum remains unchanged. A switch to vacuum II is now no longer possible. It is also not possible to switch down from level II to level I if step I has been completed. Pressing the switch (9) then allows the vacuum to stop between level I and II, but not exactly in the defined area. The mixing process is only triggered when the start button (4) is pressed. The end of the mixing time is shown on the digital time display as „0:00“ and is also accompanied by an audible signal. The mixing cup continues to be held by the vacuum once the mixing time ends, until the vacuum switch (7) is brought to the 0 position. When the vacuum reduces, the mixing cup comes loose from the mixing/suction connections (6) and must be held in place. The parameters entered - vacuum, rotational speed and mixing time - remain saved until a new setting is entered, even if the main switch (5) is turned off and on again.

Digital time display

The required time setting for the mixing process is adjusted using the knob (3). An accurate setting of 1-60 s is possible up to one minute, an accurate setting of up to 9 min/50 s max. is possible over one minute. When the switch (4) is pressed, the mixing process commences and the time runs backwards to zero. If the switch (4) is pressed again during the mixing process, the mixing process is interrupted and the display is automatically set back to the time set. When the time set lapses, „0:00“ appears on the display (2) and a warning signal also sounds.

5.4 Mixing cups

The conical shape of the stainless steel mixing cup ensures that the components are constantly mixed. The intermediate space between the mixing spiral (stirrer) and the cup wall is very small, thereby ensuring that the material is mixed to an optimum extent. The mixing coil of the mixing spiral has a left-hand thread, the device has a motor that runs in a clockwise direction. The material to be mixed is therefore pressed down and outwards during the mixing process and transported in the centre to the top, where a bubble-free, homogeneous material is formed in the mixing cup. Fluid residues or solid material residues are prevented due to the smooth inside surface of the mixing cup, as deep grooves, such as those which are present in plastic cups, cannot occur. Mixing cups and mixing spirals should only be cleaned out with clear water and using a commercially-available brush on material that has not yet hardened. However, if solid plaster or investment material residues are still found on the inside wall of the cup or on the mixing spiral despite this, these can be removed using the plaster dissolution speed REF 52001010.

5.5 Filling level of the mixing cups

The mixing cups are available in various sizes.

These can be filled to varying levels depending on the material.

Cup size in ccm	max. filling level in g	Investment material	Plaster	Silicone
50	50	✓	✓	✓
250	200	✓	✓	✓
750	600	✓	✓	✓
1000	900	✓	✓	✗
	600	✗	✗	✓

6. Cleaning and maintenance

6.1 Cleaning

Do not clean the device with compressed air!

Only clean from the outside using a lightly dampened cloth with the mains plug unplugged. Ensure that electronic parts do not come into contact with moisture. Do not use any cleaning products containing solvents!

6.2 Maintenance

Clean the holder for the mixing cup at the unit, the holder of the cup and the seal as well as the prefilter according to the level of contamination. The prefilter is located in the mixing/suction connections (6). Switch the device off and remove the mains plug before changing the prefilter. The prefilter is unfastened by turning it in an anti-clockwise direction with the help of a size 15 fork wrench and is then cleaned in an ultrasonic bath.

It is then screwed tight again by turning it in a clockwise direction. We recommend changing the prefilter every 1 to 2 years. The prefilter should not just be cleaned once the vacuum takes longer to build up, but at regular intervals, depending on how much the device is used.

CAUTION

Never operate the device without a prefilter.

The O-ring on the mixing/suction connections (6) and on the mixing spiral should be kept clean in order to guarantee an optimum vacuum. The vacuum pump does not require maintenance.

7. Troubleshooting

Fault	Cause	Troubleshooting
Device switches off!	Motor is overloaded	Restart! Turn device „off“ and „on“ again.
Device does not start/ work! LED display does not illuminate!	Network data and/or connections, parameters do not match	- Compare data with type-plate - Insert mains plug - Switch mains switch (5) to „on“ - Set knob (3) to the recommended mixing time (> 0 s)
	Microfuse faulty	Replace microfuse T 2.0 [A]
No vacuum build up	Vacuum pump defective	Contact bredent and send device in if necessary
	Seals are contaminated or porous	- Keep seals on the mixing spiral and mixing/ suction connections clean - Exchange porous seals
Vacuum builds up but mixing cup does not stay in place	Drilled hole in the lid of the mixing spiral is blocked	Clean the plastic insert
	Prefilter blocked	Disassemble and clean prefilter
Device is too loud	Sound absorber is broken or missing entirely	Remove sound absorber residues/install missing sound absorber
Mixing spiral is jerky or mixing results are not correct!	Electronic control	Contact bredent and send device in if necessary
	Mixing spiral is bent or contaminated	Clean or replace!

8. Declaration of conformity

breident GmbH & Co.KG, Weissenhomer Str. 2, 89250 Senden, Germany, hereby declares that the ecovac mixing system has been developed in accordance with the relevant provisions, European directives and harmonised standards in view of the design and construction of the product in the form brought to the market by us. The declaration of conformity can be requested if required.

9. Warranty

The ecovac mixing device has a 1-year warranty.

In case of any claims for warranty the original dealer's invoice is required. Warranty services do not result in an extension of the warranty! Faults and consequences that can be attributed to improper treatment, cleaning, maintenance and natural wear are excluded from the warranty. Any modifications or repairs to the device and its components must only be carried out by breident or by third parties with express authorisation (certification) to carry out such work, and those modifications or repairs must be carried out in line with legal engineering practice. Failure to comply with the above will result in the invalidation of the warranty! The product may only be used by trained dental staff. Only original tools and components may be used for processing. The serial number of all components in use must be documented to ensure traceability and to handle complaints. These operation and maintenance instructions are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indications described under item 1.2.

10. Verwendete Symbole

Typ	ecovac
	Batch code
	Serial number
	Year of manufacture
U/min	Speed per minute
	Fuse T 2.0 [A]
	Manufacturer
	Symbol for „European Conformity“
	Follow the instructions!
	Automatic start up - moving parts!
	Caution
	Electrical voltage warning
	Do not dispose of with household waste

11. Other information

The information contained in these instructions for use is always updated in line with the latest results and experiences. We therefore recommend reading the instructions for use prior to using a new machine. The user is responsible for how the product is used. No liability is accepted for defective results, since breident has no influence on subsequent workmanship. Any claims for compensation may only be based on the value of our products.

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